Limited Two-Year Warranty

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English • Spanish

Limited Two-Year Warranty

Simple Diagnostics guarantees that its Digital Blood Pressure Monitor will be free from manufacturing defects under normal use for as long as the original purchaser owns the product. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Digital Blood Pressure Monitor. This warranty is voided if the Digital Blood Pressure Monitor product is misused or abused in any manner.

If the Digital Blood Pressure Monitor fails to operate during the time the original purchaser owns it, return it postage prepaid along with \$5.00 for return shipping and insurance to: Simple Diagnostics, Attn: Repair Department, PO Box 128, Williston Park, NY 11596. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. Simple will repair or replace the defective unit at the option of Simple Diagnostics. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. **Simple will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others.** Simple shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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Instruction Manual

Talking Monitor

Premium Digital Blood Pressure Wrist Monitor

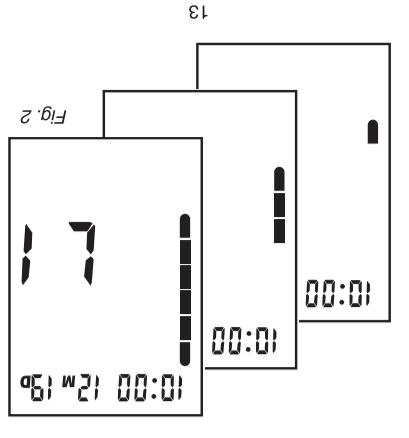
SDI-1086WT

Please read this instruction manual completely before operating this unit.

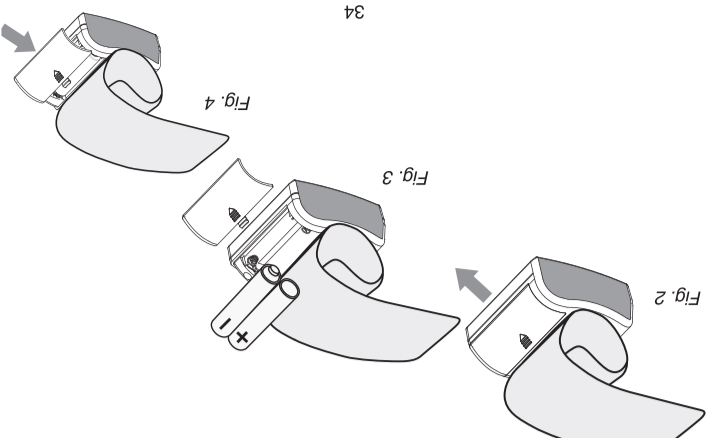
English • Spanish

Voice Language Setting

- After the desired language has been selected, you can adjust the volume by pressing and holding down the MEMORY button until the volume reaches the desired level, then release the MEMORY button to set.
- Continue to hold the MEMORY button down until the volume reaches the desired level, then release the MEMORY button to set.
- The unit will automatically turn off after the desired volume level has been set.

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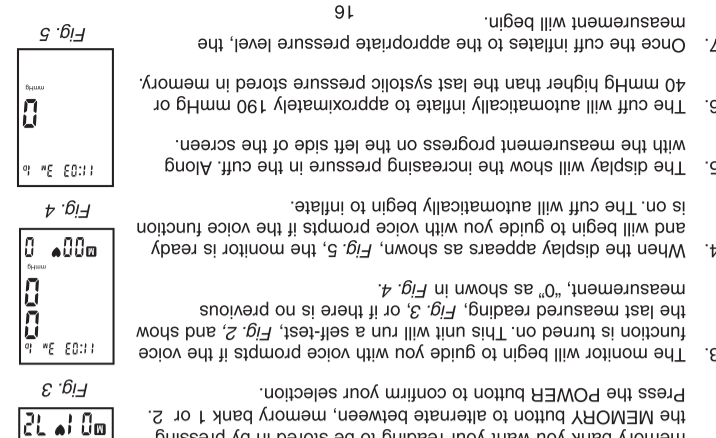
Taking Your Blood Pressure Reading

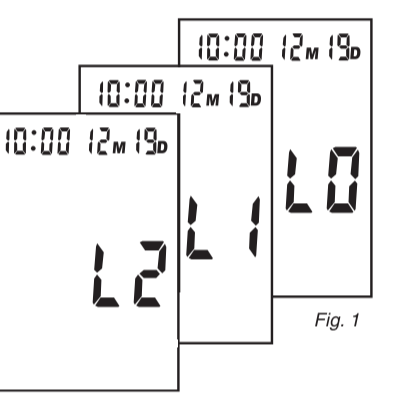
- Rest your elbow on a solid surface with your palm facing upward.
- Place your hand so that the cuff is at the same level as your heart. Fig. 1. Flex your left hand.
- Press the POWER button to turn the unit on and select the memory bank you want your reading to be stored in by pressing the MEMORY button to alternate between memory bank 1 or 2.
- Press the POWER button to confirm your selection.
- The monitor will begin to guide you with voice prompts if the voice function is turned on. This unit will run a self-test. Fig. 2 and show the last measured reading. Fig. 3, or if there is no previous measurement, "0" as shown in Fig. 4.
- When the display appears as shown, Fig. 5, the monitor is ready to begin to guide you with voice prompts if the voice function is on. The cuff will automatically begin to inflate.
- The display will show the increasing pressure in the cuff. Along with the measurement progress on the left side of the screen.
- The cuff will automatically inflate to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory.
- Once the cuff inflates to the appropriate pressure level, the measurement will begin.

Voice Language Setting

To set the desired voice language:

- When the unit is off, you can select the desired voice language by pressing and holding the MEMORY button.
- Now the screen will cycle through L0, L1 and L2 as shown in Fig. 1. L0 - voice function is off L1 - voice is in English L2 - voice is in Spanish The monitor will say the corresponding language with the displayed language code.
- Once the desired language has been selected, release the MEMORY button.

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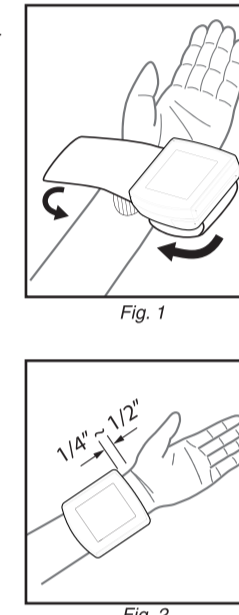
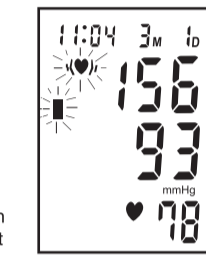
Irregular Heartbeat Detection

Your digital blood pressure monitor features Irregular Heartbeat Detection. This feature allows users to accurately monitor blood pressure even if an irregular heartbeat should occur.

When an irregular heartbeat is detected, the IHB icon will appear on the display, Fig. 1.

NOTE: Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently at the end of a reading.

Why Monitor Your Blood Pressure at Home?

Many factors such as genetics, age, sex, attitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure. Influences such as sleep or relaxation decrease blood pressure, while anxiety or exercise increases blood pressure.

A visit to a physician's office can be a stressful situation for a patient. And, anxiety is a known factor in raising blood pressure. This temporary occurrence of elevated blood pressure at the physician's office is commonly referred to as "white coat syndrome".

Whether or not you experience "white coat syndrome", home blood pressure monitoring provides you with the opportunity to supplement your physician's office measurements. These home readings, when taken over a period of time, can show an accurate indication of change. Furthermore, making important decisions in the diagnosis and treatment of your condition. Because of this, it is important to take consistent, daily measurements of your blood pressure.

The variations in your individual readings should only be interpreted by your physician or a trained health care professional.

WHO Blood Pressure Classifications

Standards for assessment of high or low blood pressure, regardless of age, have been established by the World Health Organization (WHO) as shown in the chart on the following page.

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World Health Organization (WHO) Indicator

The WHO indicator, located on the left side of the LCD screen, will identify your reading within a range in accordance with the guideline set by the World Health Organization (WHO). See the chart below for reference.

Stage	Systolic pressure	Diastolic pressure
Stage 3 Hypertension	≥180 mmHg	≥110 mmHg
Stage 2 Hypertension	160-179 mmHg	100-109 mmHg
Stage 1 Hypertension	140-159 mmHg	90-99 mmHg
High Normal	130-139 mmHg	85-89 mmHg
Normal	120-129 mmHg	80-84 mmHg
Optimal	<120 mmHg	<80 mmHg

General Blood Pressure Information

What is Blood Pressure?

Blood pressure is the pressure that is exerted by blood flowing against the walls of the blood vessels throughout your body.

Your heart, which is the center of the circulatory system, provides the force for the blood to flow or circulate. When your heart contracts or beats, the blood is forced through the blood vessels increasing the pressure. This is the highest pressure in the cycle or what is referred to as **SYSTOLIC BLOOD PRESSURE**. In between beats, your heart relaxes and your blood pressure decreases. This is referred to as **DIASTOLIC BLOOD PRESSURE**.

This complete series of events which occurs in a single heartbeat is known as the **CARDIAC CYCLE**.

Your Digital Blood Pressure Monitor will automatically read your blood pressure and display both systolic and diastolic readings on the screen. Your systolic will be positioned as the upper number and the diastolic reading will be the lower number.

(systolic) **120/80** (diastolic)

Blood pressure is measured in millimeters (mm) of Mercury (Hg) and is generally recorded with the systolic pressure (120) listed first and the diastolic pressure (80) listed second. The numbers are typically separated by a slash mark (/) as shown above.

Both pressure readings, the **SYSTOLIC** and the **DIASTOLIC**, are necessary for a physician to evaluate the status of a patient's blood pressure. Please contact your physician for specific information regarding your own blood pressure.

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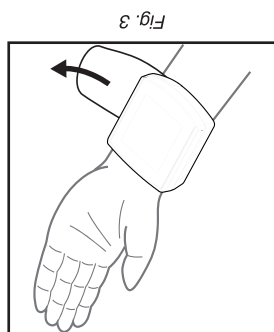
Index

Introduction	2
Product Identification and Caution	3
General Blood Pressure Information	4-6
Important Information Before Use	7
World Health Organization (WHO) Indicator	8
Irregular Heartbeat Detection	9
Battery Installation/Replacement	10
Setting the Date/Time	11-13
Voice Language Setting	12
Applying Your Blood Pressure Cuff	14-15
Taking Your Blood Pressure Reading	16-17
Recalling Measurements in Memory	18
Description of Display Symbols	19-20
Troubleshooting	21
Care and Maintenance	21
Product Specifications	22
Sample Blood Pressure Log	23
Blood Pressure Log	24

6. Wrap and fasten the cuff securely, yet comfortably around your wrist. Fig. 3.

NOTE: The left wrist should be used unless you are unable to or should not use your arm or wrist. In consultation with your physician, you may modify the following instructions and apply the cuff to your right wrist.

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Fig. 1

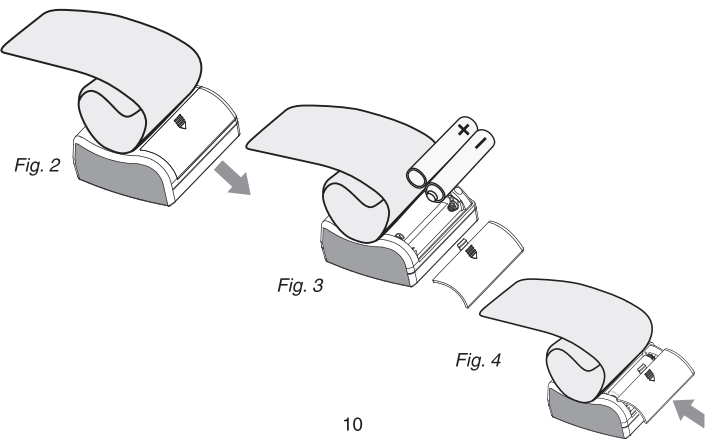
Battery Installation/Replacement

Replace the batteries when the Low Battery symbol appears on the display, Fig. 1, or when the display remains blank when the POWER Button is pressed.

To change the batteries:

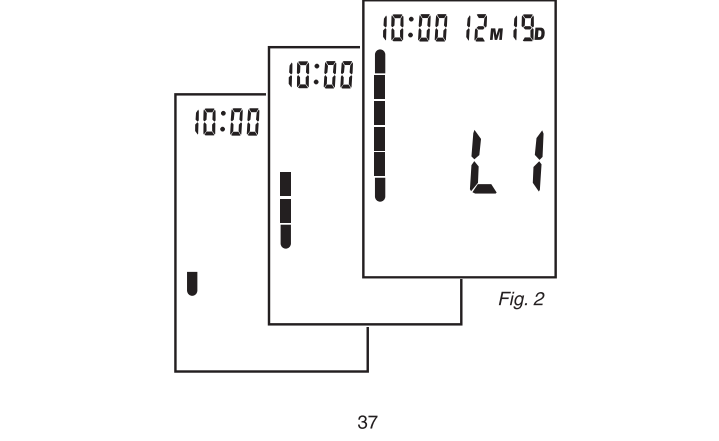
1. Open the battery cover, Fig. 2.
2. Making sure the battery polarity is correct, install two AAA alkaline batteries, Fig. 3.
3. Close the battery cover, Fig. 4.

NOTE: It is recommended to remove the batteries if the unit will not be used for an extended period of time.

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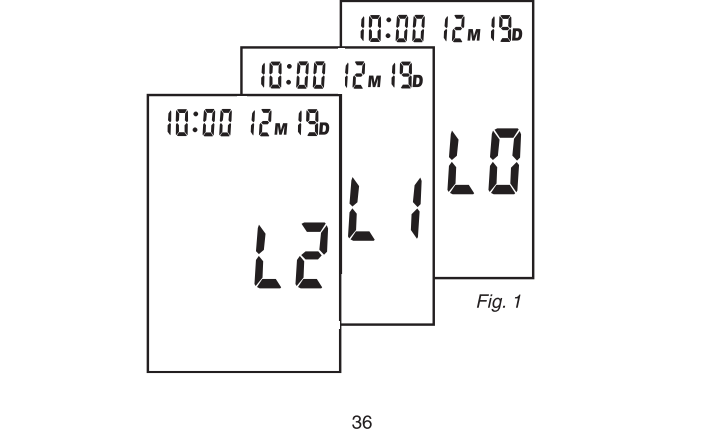
Setting the Date/Time

NOTE: When the unit is off, the date and time will still appear at the top of the screen.

1. When the unit is off, press and release both the POWER and MEMORY button at the same time. The month will begin to flash, Fig. 1.
2. Press the MEMORY button to change until the desired number appears.

NOTE: To scroll through the numbers quicker, hold down the MEMORY button.

3. Press the POWER button to set the entry.

4. The date, Fig. 2, followed by the hour and minutes will blink. Repeat steps 2 and 3 to adjust the date/time setting.

5. After setting the minutes, the unit will automatically exit out of the date/time setting mode, Fig. 3.

NOTE: The time is expressed in 24hr military time. The chart below explains the time conversion.

Regular Time (12 hours)	Military Time (24 hours)	Regular Time (12 hours)	Military Time (24 hours)
12 am	0	12 pm	12
Midnight		Noon	
1 am	1	1 pm	13
2 am	2	2 pm	14
3 am	3	3 pm	15
4 am	4	4 pm	16
5 am	5	5 pm	17
6 am	6	6 pm	18
7 am	7	7 pm	19
8 am	8	8 pm	20
9 am	9	9 pm	21
10 am	10	10 pm	22
11 am	11	11 pm	23

While taking a measurement, you can stop the inflation or deflation process of the cuff at any time by pressing the POWER button.

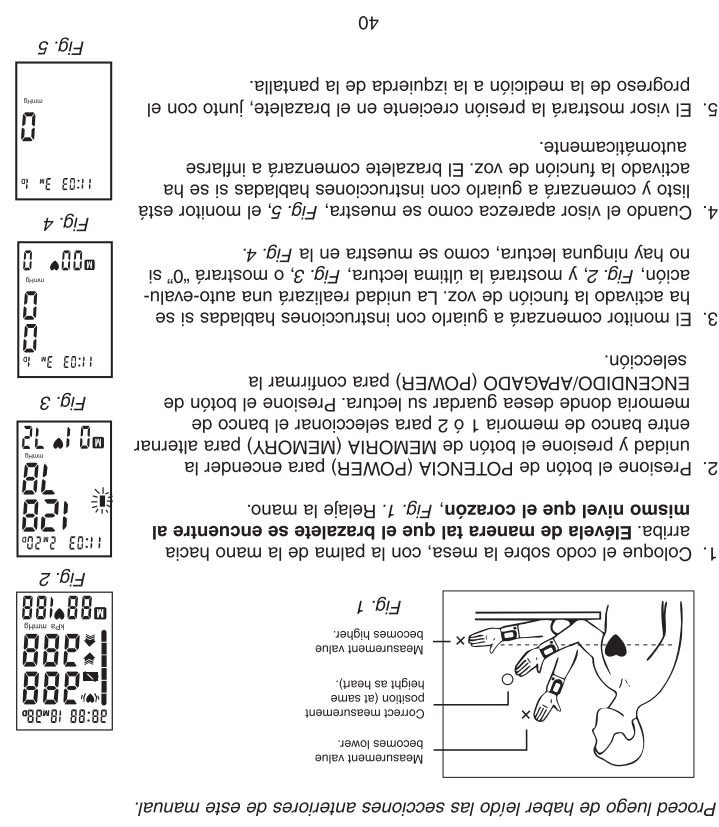
12. This device is intended for adult use. healthcare professional.
11. Although such cases are rare, for those with an extremely weak pulse or irregular pulse, errors may result which prevent proper measurement. If abnormal variations are noticed, consult with your physician or trained professional.
10. Wait a minimum of 15 minutes between readings to allow for the blood vessels to return to normal. The wait time may vary depending on your individual physiological characteristics.
9. Record your daily readings on the chart provided.
8. Take your readings at the same time, each day (or as recommended by your physician) to get an accurate indication of change in your blood pressure.
7. Keep the monitor stable during measurement to achieve an accurate reading. Remain still; do not talk during the measurement.
6. Apply the cuff to the proper position on the left wrist. The unit should be located on the inside of your left wrist with the display panel in view.
5. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
4. Rest 15 minutes prior to taking a reading.
3. Avoid smoking, eating, taking medication, alcohol consumption or physical stress, avoid taking your measurement until the feeling subsides.
2. Perform your measurement in a quiet place. You should be seated in a relaxed position.
1. Through regular use of this device and recording of your measurements, you can keep your physician informed of the changes in your blood pressure. A trained health care professional who is familiar with your medical history, or a physician or blood pressure measurements should only be interpreted by a physician or

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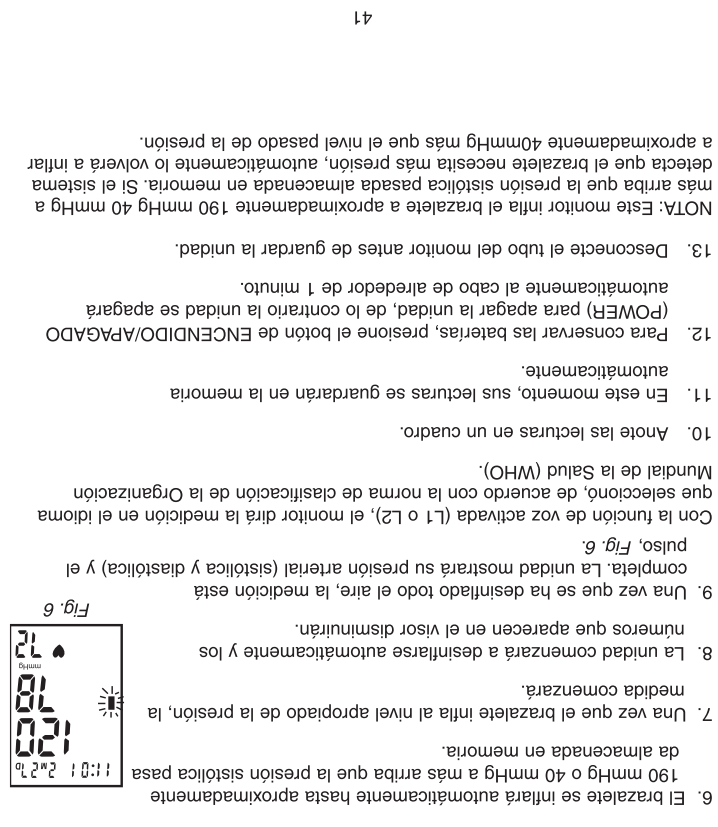
Introduction

To achieve the maximum benefit from your blood pressure monitor, we recommend that you first consult with your physician or trained health care professional.

Thank you for purchasing an Automatic Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

The method of measurement that your Automatic Monitor uses is called the oscillometric method. The monitor detects your blood's movement through the artery in your wrist and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor easy-to-use. The preformed cuff requires little effort to apply and is comfortable to wear.

Clinical research has proven a direct relationship between blood pressure in the wrist and blood pressure in the arm. Because the arteries in the wrist and arm are connected, changes in wrist blood pressure reflect changes in arm blood pressure. Frequent wrist blood pressure measurements will provide you and your physician with an accurate indication of change in your true blood pressure.

People with severe hypertension, severe arteriosclerosis or severe diabetes should consult their physician about monitoring blood pressure at the wrist.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

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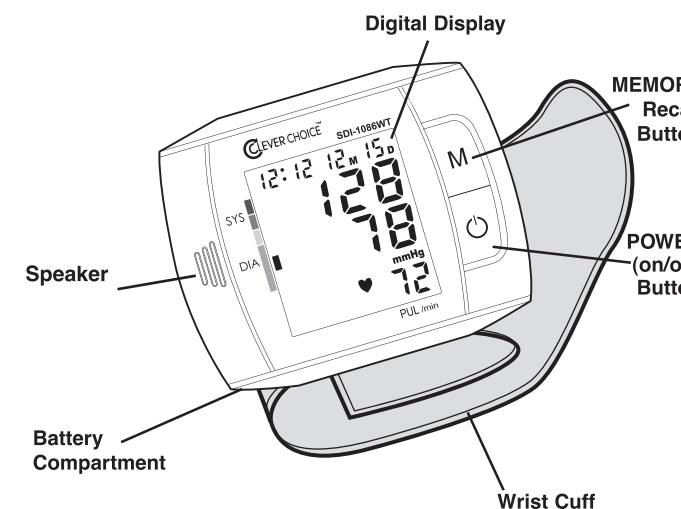
Product Identification and Caution

CAUTION
While taking a measurement, you can stop the inflation or deflation process of the cuff at any time by pressing the POWER button.

Description of Display Symbols

Indicates a measurement within Stage 1 Hypertension.

Indicates a measurement within Stage 2 Hypertension.

Indicates a measurement within Stage 3 Hypertension.

Appears when batteries should be replaced.

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Recalling Measurements in Memory

You can recall up to 120 total measurements in memory (60 per memory bank). These measurements can be shared with your physician or trained healthcare professional.

1. Press and release the MEMORY button. The unit will display either L1 or L2 depending upon the user bank selected. It will also show how many measurements are stored within that bank.

2. To select the appropriate user bank press the POWER button to switch between L1 and L2.

3. Once the desired user memory bank is selected, press the MEMORY button again to scroll through all the readings. After you've scrolled through all the readings, the unit will automatically turn off.

NOTE: Each memory bank can store up to 60 readings. When the number of readings exceeds 60, the oldest data will be replaced with the new record.

2. Select the desired user bank by pressing and releasing the MEMORY button.

3. Press and hold down the MEMORY button until two horizontal lines flash. This indicates that all measurements within the selected memory bank have been erased.

Product Specifications

Name and Model Number Talking Monitor-Premium Digital Blood Pressure Wrist Monitor, SDI-1086WT

Display System Digital display/LCD

Measuring Method Oscillometric Method

Power Source Automatic Air Inflation and Measurement

Measuring Range 2 AAA batteries

Accuracy Pressure: 0 to 300 mmHg

Pressurization Pulse: ±5% of reading

Deflation Automatic pressurization by pump

Memory Automatic pressure release valve

Automatic Power Off Built-in memory enabling display of up to 120 (60 per memory bank) measurements

Battery Life Approximately 1 minute after last button operation

Operation Environment Temperature 41°F – 104°F (5°C – 40°C)

Storage Environment Humidity <85% RH max

Weight Temperature -4°F – 131°F (-20°C – 55°C)

Dimensions Humidity <95% RH max

Wrist Circumference Range 4.6 oz. (without batteries)

Accessories 3-1/4" x 2-3/4" x 1-1/8"

Specifications are subject to change without notice. Simple Diagnostics

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Care and Maintenance

1. Only use a soft, dry cloth to clean your blood pressure monitor.

2. Avoid using any types of liquids on the monitor or cuff.

3. Store your monitor in the provided storage case when not in use.

4. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.

5. Never disassemble the monitor or cuff.

6. Dropping or subjecting your blood pressure monitor to strong shocks should be avoided.

7. Remove batteries prior to storing.

Troubleshooting

If any abnormality occurs during use, please check and correct the following:

Correction

Display is blank when power is on. Measurements are different from those typically measured by physician or every measurement reading is different.

Measurements are different from those typically measured by physician or every measurement reading is different.

Review and follow "Applying Your Blood Pressure Cuff" and "Taking Your Blood Pressure Reading" sections. Replace the batteries.

Remember, blood pressure readings are influenced by physical and mental conditions and/or even the time of day. Daily measurement should be interpreted by your physician.

Do not talk or move during the measurement. Review "Applying Your Blood Pressure Cuff" section.

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Description of Display Symbols

Shows the pulse rate per minute.

Occurs when an error was made during measurement.

Appears when there is an irregular heartbeat detected during measurement

Appears when the cuff is unstable or it has too much air left in it. Once the arrow stops flashing, the unit is ready to take a measurement.

Appears when a measurement error has occurred. Restart the measurement.

Appears when recalling a measurement from memory

Indicates the memory user's memory bank

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8. The unit will begin to automatically deflate and the numbers appearing on the display will decrease.

9. When all the air is deflated, the measurement is complete. The unit will display your blood pressure (systolic and diastolic) measurements and pulse rate, Fig. 6.

With the voice function on (L1 or L2), the monitor will say the measurement with the language you selected according to the classification standard of WHO (World Health Organization).

10. Record your readings on a chart.

11. At this point, your measurements will be automatically saved in memory.

12. To conserve the batteries, press the POWER button to turn the power off. Otherwise, the unit will automatically shut off after approximately 1 minute.

NOTE: This monitor inflates the cuff to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory. If the system detects that the cuff needs additional pressure, it will automatically reinflate to approximately 40mmHg higher than the last pressure level.

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