Limited Two-Year Warranty

Simple Diagnostics guarantees that its Digital Blood Pressure Monitor will be free from manufacturing defects under normal use for as long as the original purchaser owns the product. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Digital Blood Pressure Monitor. This warranty is voided if the Digital Blood Pressure Monitor is misused or abused in any manner.

If the Digital Blood Pressure Monitor fails to operate during the time the original purchaser owns it, return it postage prepaid along with \$5.00 for return shipping and insurance to: Simple Diagnostics, Attn: Repair Department, PO Box 128, Williston Park, NY 11596. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. Simple will repair or replace the defective unit at the option of Simple Diagnostics. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

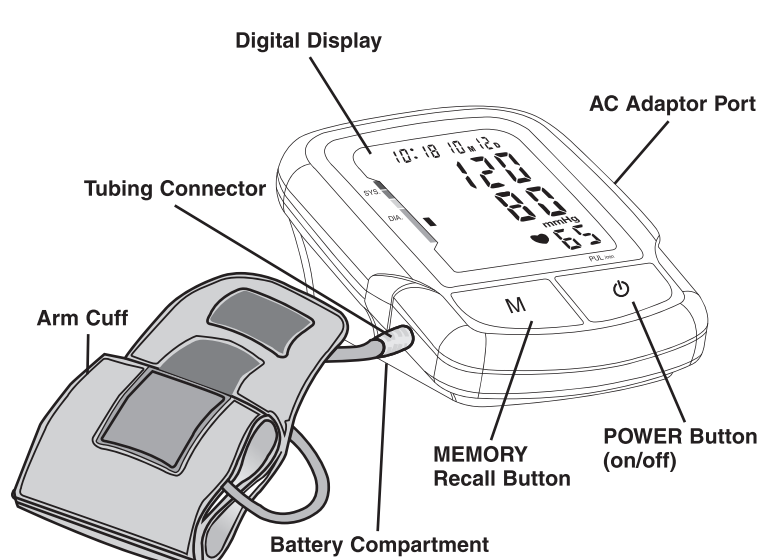
ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. Simple will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. Simple shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

PO Box 128, Williston Park, NY 11596

Product Identification & Caution

CAUTION:
You can stop the inflation or deflation process anytime by pressing the Power button.

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Why Monitor Your Blood Pressure at Home?

Many people experience "White Coat Syndrome", which is a temporary elevation of blood pressure during a visit at their physician's office. Although only temporary, it can cause blood pressure to rise and give a false indication of a person's true blood pressure.

- Whether or not you experience "White Coat Syndrome", home blood pressure monitoring provides:
- 1) The opportunity to supplement your physician's office measurements.
 - 2) An accurate indication of any change.
 - 3) A record to assist your physician in evaluating your health and making important decisions.
 - 4) Consistent, daily measurements of your blood pressure.

NOTE: Variations in your individual readings should only be interpreted by your physician or trained healthcare professional.

General Blood Pressure Information

What is Blood Pressure?
Blood pressure is the pressure that is exerted by blood flowing against the walls of the blood vessels throughout your body.

Your heart, which is the center of the circulatory system, provides the force for the blood to flow or circulate. When your heart contracts or beats, the blood is forced through the blood vessels increasing the pressure. This is the highest pressure in the cycle and what is referred to as SYSTOLIC blood pressure. In between beats, your heart relaxes and your blood pressure decreases. This is referred to as DIASTOLIC blood pressure.

This complete series of events, which occurs in a single heartbeat, is known as the CARDIAC CYCLE.

Your Simple Automatic Blood Pressure Monitor will automatically read your blood pressure and display both systolic and diastolic readings on the screen. Your systolic will be positioned as the upper number and the diastolic reading will be the lower number.

(systolic)
120/80
(diastolic)

World Health Organization (WHO) Indicator

The WHO Indicator, located on the left side of the display, will identify your reading within a range in accordance with the guidelines set by the World Health Organization (WHO). See the chart below for reference.

| Category | Systolic pressure | Diastolic pressure |
|-----------------------------|-------------------|--------------------|
| Stage 3 Hypertension | >180 mmHg | >110 mmHg |
| Stage 2 Hypertension | 160-179 mmHg | 100-109 mmHg |
| Stage 1 Hypertension | 140-159 mmHg | 90-99 mmHg |
| High Normal | 130-139 mmHg | 85-89 mmHg |
| Normal | 120-129 mmHg | 80-84 mmHg |
| Optimal | <120 mmHg | <80 mmHg |

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Irregular Heartbeat Detection

Your digital blood pressure monitor features Irregular Heartbeat Detection. This feature allows users to accurately monitor blood pressure even if an irregular heartbeat should occur.

When an irregular heartbeat is detected, the IHB icon will appear on the display, Fig. 1.

NOTE: Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently at the end of a reading.

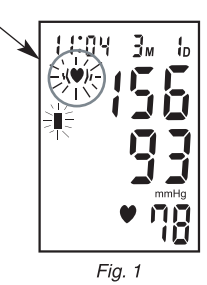
Fig. 1

Applying Your Blood Pressure Cuff

Before applying your blood pressure cuff, be sure you have selected the appropriate cuff:
Small Adult Cuff fits arm circumference 9-5/8" - 11-3/4" (22-30 cm)
Adult Cuff fits arm circumference 11-3/4" - 16-1/2" (30-42 cm)
Large Adult Cuff fits arm circumference 16-1/2" - 18-7/8" (42-48 cm)

NOTE: If for any reason you are unable to or should not use your left arm, please modify the following instructions and apply the cuff to your right arm. Your physician can tell you which arm is best for you to use.

1. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
2. Be seated with your feet on the floor.
3. Position the cuff on a solid surface with the tubing facing UP and AWAY from you, Fig. 1. The metal bar on the cuff should be to the left of the tubing.
4. Widen or open the cuff by pulling or rolling the bottom of the cuff towards the right, Fig. 2. This should open the cuff, creating a cylinder. Do not extend the cuff beyond the metal bar.
5. Insert your arm into the cuff (cylinder). Position the (D) mark over the main artery (on the inside of your arm), Fig. 3.
6. The bottom edge of the cuff should be positioned approximately one inch ABOVE the elbow joint, Fig. 4.
7. Reaching underneath your left arm with your right hand, pull the end of the cuff towards your body to tighten the cuff, Fig. 5. Wrap and secure the cuff making sure that the (D) mark remains as shown, Fig. 3.
8. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert two fingers between your arm and the cuff.


NOTE: The cuff size is suitable for use when the vertical "index" mark is within the horizontal "OK" range. A different sized cuff is needed if the "index" mark is outside of the "OK" range.

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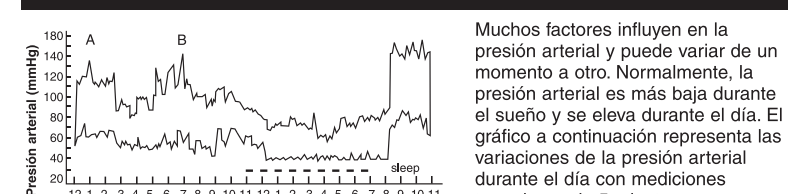Taking Your Blood Pressure Reading

Proceed only after reading the previous sections of this manual.

1. Position the monitor on a flat, stable surface with the digital display panel in view. Insert the cuff tubing connector into the left side of the monitor, Fig. 1.
2. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig. 2. Relax your left hand.
3. Press the POWER button to turn the unit on and select the memory bank you want your reading to be stored in by pressing the MEMORY button to alternate between memory bank 1 or 2. Press the POWER button to confirm your selection.

4. This unit will run a self-test, Fig. 3, and show the last measured reading, Fig. 4, or if there is no previous measurement, "0" as shown in Fig. 5.

5. When the display appears as shown, Fig. 6, the monitor is ready. The cuff will automatically begin to inflate.

6. The display will show the increasing pressure in the cuff. Along with the measurement progress on the left side of the screen.

7. The cuff will automatically inflate to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory.

8. Once the cuff inflates to the appropriate pressure level, the measurement will begin.

9. The unit will begin to automatically deflate and the numbers appearing on the display will decrease.

10. When all the air is deflated, the measurement is complete. The unit will display your blood pressure (systolic and diastolic) measurements and pulse rate, Fig. 7.

Care and Maintenance

1. Disconnect the cuff tubing from the monitor and remove batteries prior to storing.
2. Only use a soft, dry cloth to clean your blood pressure monitor.
3. Avoid using any types of liquids on the monitor or cuff.
4. Do not store the unit where it will be exposed to direct sunlight, dust or humidity.
5. Avoid extreme temperatures.
6. Never disassemble the monitor or cuff.
7. Dropping or subjecting your blood pressure monitor to strong shocks should be avoided.

Product Specifications

Name and model number: Automatic Premium Digital Blood Pressure Arm Monitor, SDI-786A

Display system: Digital display/LCD

Measuring method: Oscillometric

Power source: 4 "AA" batteries

Measuring range: Pressure: 0-300 mmHg
Pulse: 30-180 beats/minute

Accuracy: Pressure: ±3 mmHg
Pulse: ±5% of reading

Inflation/Deflation: Pump Driven
Automatic Pressure Release Valve

Memory: Built-in memory enabling display of up to 120 (60 per memory bank) measurements

Automatic shut-off: Approximately 1 minute after last button operation

Operation environment: Temperature 41°F - 104°F (5°C - 40°C)
Humidity < 85%

Storage environment: Temperature 41°F - 131°F (-20°C - 55°C)
Humidity < 95%

Monitor dimensions: 4-1/4" x 6" x 2-3/8"

Weight: 10.5 oz. (without batteries)

Arm circumference range: Adult size cuff fits arm circumference: 11-3/4" - 16-1/2" (30-42 cm), Cuff, detailed guidebook, quick start guide, blood pressure log, storage case

Options (sold separately): Large adult size cuff fits arm circumference: 16-1/4" - 18-7/8" (42-48 cm), Small adult size cuff fits arm circumference: 9-5/8" - 11-3/4" (22-30 cm), AC Adapter

Specifications are subject to change without notice. Simple Diagnostics

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