**INSTRUCTION MANUAL**

**Automatic**  
Premium Digital Blood Pressure Arm Monitor

SDI-786A

Please read this instruction manual completely before operating this unit.

English • Spanish

Blood pressure is measured in millimeters (mm) of mercury (Hg) and is generally recorded with the systolic pressure (120) listed first and the diastolic pressure (80) listed second. The numbers are typically separated by a slash mark (/) as shown.

Both pressure readings, the SYSTOLIC and DIASTOLIC, are necessary for a physician to evaluate the status of a patient's blood pressure.

Please contact your physician for specific information regarding your own blood pressure.

**What Influences Blood Pressure?**

Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure. Influences such as sleep or relaxation decrease blood pressure, while anxiety or exercise increase blood pressure.

**WHO Blood Pressure Classifications**

Standards for assessment of high or low blood pressure have been established by the World Health Organization (WHO) as shown on the following chart:

CATEGORY	SYSTOLIC (mmHg)	DIASTOLIC (mmHg)	COLOR INDICATOR
Optimal	<120	<80	GREEN
Normal	120-129	80-84	GREEN
Stage 1 - Mild Hypertension	130-139	85-89	GREEN
Borderline Hypertension	140-159	90-99	YELLOW
Stage 2 - Moderate Hypertension	160-179	100-109	ORANGE
Stage 3 - Severe Hypertension	≥180	≥110	RED
Isolated Systolic Hypertension	≥140	<90	YELLOW
	140-159	<90	ORANGE
	160-179	<90	ORANGE
	≥180	<90	RED

This chart is only a general guideline. Contact your physician or trained healthcare professional to determine your NORMAL blood pressure.

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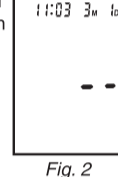
**Index**

Introduction . . . . . 3  
 Product Identification & Caution . . . . . 4  
 Why Monitor Your Blood Pressure at Home? . . . . . 5  
 General Blood Pressure Information . . . . . 5  
 Important Information Before Use . . . . . 7  
 World Health Organization (WHO) Indicator . . . . . 8  
 Irregular Heartbeat Detection . . . . . 9  
 Battery Installation/Replacement . . . . . 10  
 Setting the Date/Time . . . . . 11  
 Applying Your Blood Pressure Cuff . . . . . 12  
 Taking Your Blood Pressure Reading . . . . . 13  
 Recalling Measurements in Memory . . . . . 14  
 Description of Display Symbols . . . . . 15  
 Troubleshooting . . . . . 16  
 Care and Maintenance . . . . . 16  
 Product Specifications . . . . . 16  
 Sample Blood Pressure Log . . . . . 17

**Introduction**

To achieve the maximum benefit from your blood pressure monitor, we recommend that you first consult with your physician or trained healthcare professional.

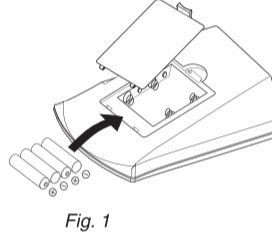
Thank you for purchasing a Simple Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

The method of measurement that your Automatic Monitor uses is called the oscillometric method. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor easy to use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

**Battery Installation/Replacement**

- Press down on tab and lift the cover.
- Insert or replace 4 "AA" batteries into the compartment, matching the indicated polarity symbols, Fig. 1.
- To close, match up and connect the bottom hooks of the cover, then push the top end.

NOTE: Replace the batteries when the Low Battery symbol appears on the display or when the display remains blank when the POWER button is pressed.


It is recommended to remove the batteries if the unit will not be used for an extended period of time.

**AC Power Adapter (optional)**

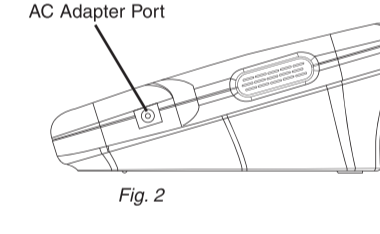An AC adapter is available separately for battery-free operation. The AC adapter plugs into the side of the unit as shown, Fig. 2. Please remember to practice electrical safety precautions while using the adapter.

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
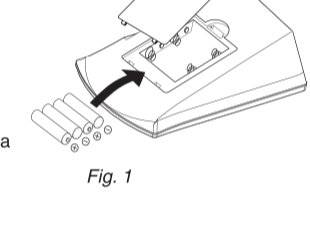
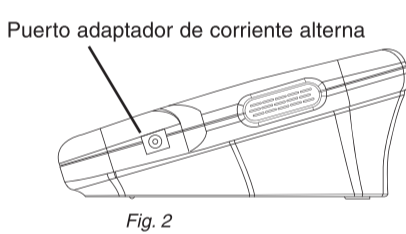
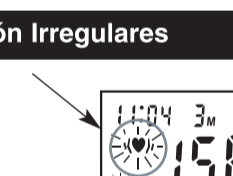
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
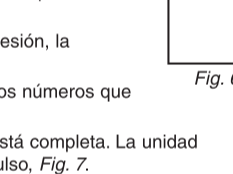
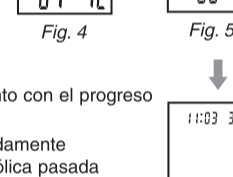
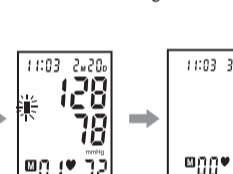
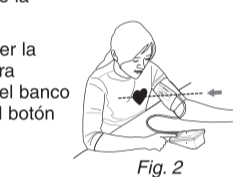
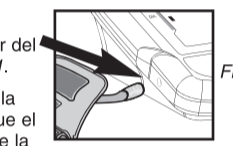
**Variations in Blood Pressure**

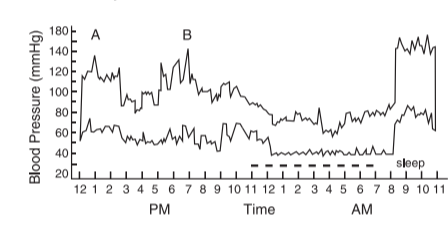Blood pressure is influenced by many factors and can change from moment to moment. Normally, blood pressure is lowest during sleeping periods and rises during the day. The graph below represents variations in blood pressure shown over a day with measurements taken every 5 minutes.

The dotted line represents the sleep period. The rise in blood pressure at 1 p.m. (A in the graph) corresponds to a stressful occurrence and at 7 p.m. (B in the graph) a period of exercise.

**Important Information Before Use**

- Blood pressure readings should be interpreted by a physician or trained healthcare professional, who is familiar with your medical history.
- Perform your measurement in a quiet place. You should be seated and relaxed.
- Avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading. If you are exhibiting signs of stress, avoid taking your measurement until the feeling subsides.
- Rest 15 minutes prior to taking a reading.
- Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
- Keep the monitor stable during measurements. Remain still; do not talk during measurements.
- Record your daily readings on the chart provided.
- Take your readings at the same time, each day (or as recommended by your physician).
- Wait a minimum of 15 minutes between readings. The wait time may vary depending on individual physiological characteristics.
- This device is intended for adult use.
- The inflation or deflation process can be stopped by pressing the POWER button.

- Record your readings on a chart.
- At this point, your measurements will be automatically saved in memory.
- To conserve the batteries, press the POWER button to turn the power off. Otherwise, the unit will automatically shut off after approximately 1 minute.
- Disconnect the tubing from the monitor prior to storing.

NOTE: This monitor inflates the cuff to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory. If the system detects that the cuff needs additional pressure, it will automatically reinflate to approximately 40mmHg higher than the last pressure level.

**Recalling Measurements in Memory**

You can recall up to 120 total measurements in memory (60 per memory bank). These measurements can be shared with your physician or trained healthcare professional.

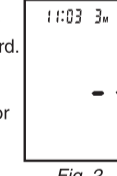
- Press and release the MEMORY button. The unit will display either U1 or U2 depending upon the user bank selected. It will also show how many measurements are stored within that bank, Fig. 1.
- To select the appropriate user bank press the POWER button to switch between U1 and U2.
- Once the desired user memory bank is selected, press the MEMORY button again to scroll through the saved measurements. After you've scrolled through all the readings, the unit will automatically turn off.

NOTE: Each memory bank can store up to 60 readings. When the number of readings exceeds 60, the oldest data will be replaced with the new record.

**To erase all stored measurements:**

- Press and release the MEMORY button. The unit will display either U1 or U2 depending upon the user bank selected and how many measurements are stored within that bank.
- Select the desired user bank by pressing and releasing the MEMORY button.
- Press and hold down the Memory button until two horizontal lines flash. This indicates that all measurements within the selected memory bank have been erased.

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
**Setting the Date/Time**

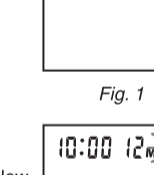NOTE: When the unit is off, the date and time will still appear at the top of the screen.

- When the unit is off, press and release both the POWER and MEMORY button at the same time. The month will begin to flash, Fig. 1.
- Press the MEMORY button to change until the desired number appears.
- Press the POWER button to set the entry.
- The date, Fig. 2, followed by the hour and minutes will blink. Repeat steps 2 and 3 to adjust the date/time setting.
- After setting the minutes, the unit will automatically exit out of the date/time setting mode, Fig. 3.

NOTE: The time is expressed in 24hr military time. The chart below explains the time conversion.

Regular Time (12 hours)	Military Time (24 hours)	Regular Time (12 hours)	Military Time (24 hours)
12 am	0	12 pm	12
1 am	1	1 pm	13
2 am	2	2 pm	14
3 am	3	3 pm	15
4 am	4	4 pm	16
5 am	5	5 pm	17
6 am	6	6 pm	18
7 am	7	7 pm	19
8 am	8	8 pm	20
9 am	9	9 pm	21
10 am	10	10 pm	22
11 am	11	11 pm	23

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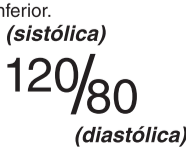
**Limited Two-Year Warranty**

Simple Diagnostics guarantees that its Digital Blood Pressure Monitor will be free from manufacturing defects under normal use for as long as the original purchaser owns the product. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Digital Blood Pressure Monitor. This warranty is voided if the Digital Blood Pressure Monitor is misused or abused in any manner.

If the Digital Blood Pressure Monitor fails to operate during the time the original purchaser owns it, return it postage prepaid along with \$5.00 for return shipping and insurance to: Simple Diagnostics, Attn: Repair Department, PO Box 128, Williston Park, NY 11596. When returning a product, please also include a letter with your name, address, phone number and brief description of the specific problem. Please carefully package the product to avoid any damage that may occur while in the transit. Packaging insurance with returned receipt is recommended. Simple will repair or replace the defective unit at the option of Simple Diagnostics. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

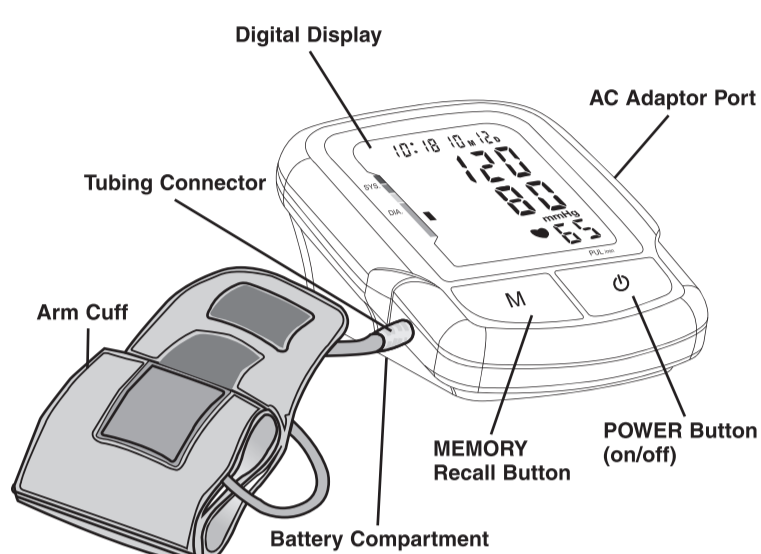
ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. Simple will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. Simple shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

PO Box 128, Williston Park, NY 11596

**Product Identification & Caution**

**CAUTION:**  
You can stop the inflation or deflation process anytime by pressing the Power button.

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**Why Monitor Your Blood Pressure at Home?**

Many people experience "White Coat Syndrome", which is a temporary elevation of blood pressure during a visit at their physician's office. Although only temporary, it can cause blood pressure to rise and give a false indication of a person's true blood pressure.

Whether or not you experience "White Coat Syndrome", home blood pressure monitoring provides:

- 1) The opportunity to supplement your physician's office measurements.
- 2) An accurate indication of any change.
- 3) A record to assist your physician in evaluating your health and making important decisions.
- 4) Consistent, daily measurements of your blood pressure.

**NOTE:** Variations in your individual readings should only be interpreted by your physician or trained healthcare professional.

**General Blood Pressure Information**

**What is Blood Pressure?**  
Blood pressure is the pressure that is exerted by blood flowing against the walls of the blood vessels throughout your body.

Your heart, which is the center of the circulatory system, provides the force for the blood to flow or circulate. When your heart contracts or beats, the blood is forced through the blood vessels increasing the pressure. This is the highest pressure in the cycle and what is referred to as **SYSTOLIC** blood pressure. In between beats, your heart relaxes and your blood pressure decreases. This is referred to as **DIASTOLIC** blood pressure.

This complete series of events, which occurs in a single heartbeat, is known as the **CARDIAC CYCLE**.

Your Simple Automatic Blood Pressure Monitor will automatically read your blood pressure and display both systolic and diastolic readings on the screen. Your systolic will be positioned as the upper number and the diastolic reading will be the lower number.

(systolic)  
**120/80**  
(diastolic)

**World Health Organization (WHO) Indicator**

The WHO Indicator, located on the left side of the display, will identify your reading within a range in accordance with the guidelines set by the World Health Organization (WHO). See the chart below for reference.

WHO Indicator	Systolic Pressure	Diastolic Pressure
Stage 3 Hypertension	>180 mmHg	>110 mmHg
Stage 2 Hypertension	160-179 mmHg	100-109 mmHg
Stage 1 Hypertension	140-159 mmHg	90-99 mmHg
High Normal	130-139 mmHg	85-89 mmHg
Normal	120-129 mmHg	80-84 mmHg
Optimal	<120 mmHg	<80 mmHg

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**Irregular Heartbeat Detection**

Your digital blood pressure monitor features Irregular Heartbeat Detection. This feature allows users to accurately monitor blood pressure even if an irregular heartbeat should occur.

When an irregular heartbeat is detected, the IHB icon will appear on the display, Fig. 1.

**NOTE:** Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently at the end of a reading.

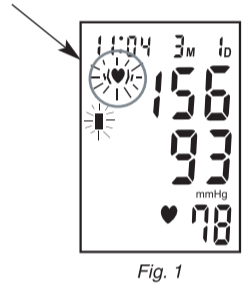
Fig. 1

**Applying Your Blood Pressure Cuff**

Before applying your blood pressure cuff, be sure you have selected the appropriate cuff:  
**Small Adult Cuff** fits arm circumference 9-5/8" - 11-3/4" (22-30 cm)  
**Adult Cuff** fits arm circumference 11-3/4" - 16-1/2" (30-42 cm)  
**Large Adult Cuff** fits arm circumference 16-1/2" - 18-7/8" (42-48 cm)

**NOTE:** If for any reason you are unable to or should not use your left arm, please modify the following instructions and apply the cuff to your right arm. Your physician can tell you which arm is best for you to use.

1. Remove any constrictive clothing or jewelry that may interfere with the cuff placement.
2. Be seated with your feet flat on the floor.
3. Position the cuff on a solid surface with the tubing facing UP and AWAY from you, Fig. 1. The metal bar on the cuff should be to the left of the tubing.
4. Widen or open the cuff by pulling or rolling the bottom of the cuff towards the right, Fig. 2. This should open the cuff, creating a cylinder. Do not extend the cuff beyond the metal bar.
5. Insert your arm into the cuff (cylinder). Position the (D) mark over the main artery (on the inside of your arm), Fig. 3.
6. The bottom edge of the cuff should be positioned approximately one inch ABOVE the elbow joint, Fig. 4.
7. Reaching underneath your left arm with your right hand, pull the end of the cuff towards your body to tighten the cuff, Fig. 5. Wrap and secure the cuff making sure that the (D) mark remains as shown, Fig. 3.
8. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert two fingers between your arm and the cuff.

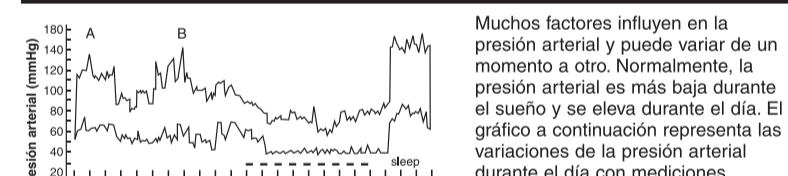
**NOTE:** The cuff size is suitable for use when the vertical "Index" mark is within the horizontal "OK" range. A different sized cuff is needed if the "Index" mark is outside of the "OK" range.

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**Taking Your Blood Pressure Reading**

Proceed only after reading the previous sections of this manual.

1. Position the monitor on a flat, stable surface with the digital display panel in view. Insert the cuff tubing connector into the left side of the monitor, Fig. 1.
2. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig. 2. Relax your left hand.
3. Press the POWER button to turn the unit on and select the memory bank you want your reading to be stored in by pressing the MEMORY button to alternate between memory bank 1 or 2. Press the POWER button to confirm your selection.
4. This unit will run a self-test, Fig. 3, and show the last measured reading, Fig. 4, or if there is no previous measurement, "0", as shown in Fig. 5.
5. When the display appears as shown, Fig. 6, the monitor is ready. The cuff will automatically begin to inflate.
6. The display will show the increasing pressure in the cuff. Along with the measurement progress on the left side of the screen.
7. The cuff will automatically inflate to approximately 190 mmHg or 40 mmHg higher than the last systolic pressure stored in memory.
8. Once the cuff inflates to the appropriate pressure level, the measurement will begin.
9. The unit will begin to automatically deflate and the numbers appearing on the display will decrease.
10. When all the air is deflated, the measurement is complete. The unit will display your blood pressure (systolic and diastolic) measurements and pulse rate, Fig. 7.

**Care and Maintenance**

1. Disconnect the cuff tubing from the monitor and remove batteries prior to storing.
2. Only use a soft, dry cloth to clean your blood pressure monitor.
3. Avoid using any types of liquids on the monitor or cuff.
4. Do not store the unit where it will be exposed to direct sunlight, dust or humidity.
5. Avoid extreme temperatures.
6. Never disassemble the monitor or cuff.
7. Dropping or subjecting your blood pressure monitor to strong shocks should be avoided.

**Product Specifications**

**Name and model number:** Automatic Premium Digital Blood Pressure Arm Monitor, SDI-786A

**Display system:** Digital display/LCD

**Measuring method:** Oscillometric

**Power source:** 4 "AA" batteries

**Measuring range:** Pressure: 0-300 mmHg  
Pulse: 30-180 beats/minute

**Accuracy:** Pressure: ±3 mmHg  
Pulse: ±5% of reading

**Inflation/Deflation:** Pump Driven  
Automatic Pressure Release Valve

**Memory:** Built-in memory enabling display of up to 120 (60 per memory bank) measurements

**Automatic shut-off:** Approximately 1 minute after last button operation

**Operation environment:** Temperature 41°F - 104°F (5°C - 40°C)  
Humidity < 85%

**Storage environment:** Temperature 41°F - 131°F (-20°C - 55°C)  
Humidity < 95%

**Monitor dimensions:** 4-1/4" x 6" x 2-3/8"

**Weight:** 10.5 oz. (without batteries)

**Arm circumference range:** Adult size cuff fits arm circumference: 11-3/4" - 16-1/2" (30-42 cm).  
Cuff, detailed guidebook, quick start guide, blood pressure log, storage case

**Options (sold separately):** Large adult size cuff fits arm circumference: 16-1/4" - 18-7/8" (42-48 cm). Small adult size cuff fits arm circumference: 8-5/8" - 11-3/4" (22-30 cm). AC Adapter

Specifications are subject to change without notice. Simple Diagnostics

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